

Copies of this poster are available from the Infection Prevention and Control Team at NHS Sheffield on Tel: 0114 305 1115.

BIOHAZARD SPILL PAK INSTRUCTIONS





USE OF SPILL PAK

FOR BLOOD AND BLOOD-STAINED BODY FLUID SPILLS

BEFORE YOU START



Always wear gloves and an apron.



Ensure good ventilation when using all chlorine products.



Check Product Expiry Date and read CoSHH information on the product label.

CoSHH information Expiry date



Pour ALL Haz-Tab granules over spill.

Leave for two minutes.

Make up a 10,000 ppm chlorine solution

Add the 4 small tablets to the empty granules container and carefully fill with water to the line.





When tablets have dissolved screw down the cap and mix gently by inversion.

DO NOT SHAKE THE CONTAINER



Remove the red strip on the dispo bag and attach to the wall, or place bag near the spill. Retain backing to act as a tie later.



Collect spilt material and granules mixture using scoop and scraper and discard all into the yellow disposal bag.



Use the solution on the disposable towels to wipe splashes on walls and remove any smears.



Discard any remaining solution by flushing down the sink or sluice with extra water.



Place all used materials, gloves last, into the disposal bag. Secure with the tie.



The bag and its contents should be discarded as clinical waste.

DON'T FORGET TO WASH YOUR HANDS!

Important Points:

- **Never** use on chemical spills or mix with other chemicals such as cleaning agents.
- Chlorine products may bleach soft furnishings and corrode metal.
- Refer to the Infection Control Policy for full details on the use of these products.
- If in doubt about the use of this product, contact your Infection Control Advisor.

***Spill Pak & Haz-Tab** products are manufactured by Guest Medical Limited of Edenbridge, Kent. 01732 867466