

Copies of this poster are available from the Infection Prevention and Control Team at NHS Sheffield on Tel: 0114 305 1115.

## URINE & VOMIT SPILLS PAK INSTRUCTIONS



**NHS**  
**Sheffield**

### USE OF THE URINE & VOMIT SPILL PAK

#### BEFORE YOU START

Do not use this product if observable blood is present in the spill.....

.....blood stained spills must be disinfected and cleaned up using a chlorine-based product such as the Guest Biohazard Spills Kit.




Always wear gloves and an apron.



Always check safety information on product labels.

Sprinkle NQ64 Granules over spill until all moisture is completely absorbed.

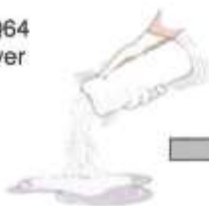

Leave for approx. 30 - 60 seconds.



Peel backing away from adhesive strip on yellow disposal bag and attach bag to wall or table. Allow to hang open.

Retain backing to act as a tie later.



Collect spilt material and granules mixture using scoop and scraper and discard all into the yellow disposal bag.

Use the surface wipes to clean and disinfect the area of the spill and remove any smears.



Place all used materials, gloves last, into the disposal bag. Secure with a tie.



The bag and its contents should be destroyed by incineration unopened.



**DON'T FORGET TO WASH YOUR HANDS!**

#### Important Points:

1. Avoid breathing dust from the granules.
2. Do not use on Blood Spills – see above .
3. Refer to your Infection Control Advisor if in doubt about using the product.

These products are manufactured by Guest Medical of Edenbridge, Kent. 01732 867 466

Visit our web site at [www.guest-medical.co.uk](http://www.guest-medical.co.uk)